

Catheter Exchange Kit


Making Foley catheter changes easier

Mediplus has worked closely with UK clinicians to design, develop and manufacture a catheter exchange kit.

The Mediplus Catheter Exchange Kit is designed to aid catheter replacement in suprapubic and difficult urethral catheterisations.

Mediplus understands that there could be many reasons why changing a catheter might present complications, so we have designed our Catheter Exchange Kit to reduce these.

The Catheter Exchange Kit includes an open tip Foley catheter (the same as that used in the Mediplus S-Cath™ System), empty syringe for balloon deflation, a 10% aqueous glycerine solution-filled syringe for balloon deflation and a 1.5 metre guidewire.

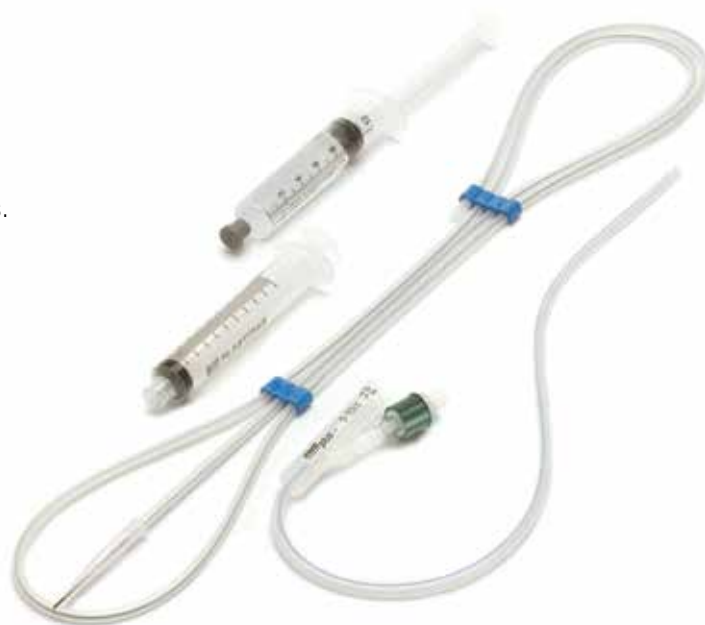

The guidewire has been manufactured with a PTFE coating to aid insertion and removal suprapubically or urethrally. The open tip Foley catheter is 100% silicone, has an integrated balloon and is smooth and round in shape. All of which improve insertion and removal.

The Mediplus Catheter Exchange Kit offers a complete solution to long-term catheterisation and complements the Mediplus S-Cath™ System.

Key Features and Benefits of the Catheter Exchange Kit

- Catheter has a soft and uniformly inflated balloon. Sits well against the bladder wall
- Smooth round shaft can minimise trauma during insertion and removal
- Use of guidewire keeps tract open
- Integrated balloon helps to minimise trauma during insertion and removal
- Catheter has radiopaque stripe for visualisation under ultrasound - X-Ray confirmation of correct positioning
- 100% silicone, Latex Free
- Designed for either suprapubic or difficult urethral use
- Colour coded catheter valve
- Manufactured in the UK



2-Way Foley catheter with open tip

The Procedure Step by Step



1. Remove urine collection device from distal end of catheter



2. Insert flexible end of two-stage guidewire into catheter drainage lumen



3. Deflate balloon of in-situ catheter using empty 10ml syringe



4. Remove catheter over guidewire making sure to leave the guidewire in position



5. Insert the new catheter over the guidewire until the bladder is reached



6. Once the catheter is within the bladder, inflate catheter balloon, via Luer valve to the volume indicated on the catheter and packaging using pre-filled syringe of glycerine solution. NB. If there is any uncertainty regarding the position of the catheter do not inflate balloon as this may harm the patient.



7. Remove the guidewire.



8. Attach suitable sterile urine drainage device to catheter.

Product Code

Description

Size

Box Quantity

Catheter Exchange Kits

5793	Silicone 2-way Foley Catheter Exchange Kit, Open Tip, with Guidewire 12Fr./Ch.	1
5795	Silicone 2-way Foley Catheter Exchange Kit, Open Tip, with Guidewire 14Fr./Ch.	1
5797	Silicone 2-way Foley Catheter Exchange Kit, Open Tip, with Guidewire 16Fr./Ch.	1
5799	Silicone 2-way Foley Catheter Exchange Kit, Open Tip, with Guidewire 18Fr./Ch.	1

Replacement Foley Catheters

5765	Silicone 2-way Foley Catheter, open tip 12Fr./Ch.	10
5767	Silicone 2-way Foley Catheter, open tip 14Fr./Ch.	10
5769	Silicone 2-way Foley Catheter, open tip 16Fr./Ch.	10
5763	Silicone 2-way Foley Catheter, open tip 18Fr./Ch.	10

* Catheter Exchange Kit includes an empty syringe for balloon deflation, 10% Aqueous glycerine solution filled syringe for balloon inflation, 1 x 2-way Foley catheter with open tip, 2-Stage Guidewire and peelable product record labels

Feb-17/02

mediplus

*Working together
to make the difference*

Mediplus Ltd

Unit 7 The Gateway Centre, Coronation Road, Cressex Business Park,
High Wycombe, Bucks, HP12 3SU UK

Tel: +44 (0)1494 551200 | Fax: +44 (0)1494 536333

Email: help@mediplus.co.uk | Web: www.mediplus.co.uk



**INVESTORS
IN PEOPLE**

Accredited
Until 2019

